

IN THE EVENT OF A RESCUE

Operating Instructions -

1. First responder should assess if the victim is responsive (breathing).
2. If not, or questionable... call 911 and start CPR.
3. Instruct someone to get the portable AED device (take black bag from AED cabinet).
- 4. IMPORTANT: Push and release on/off button to start the unit.**
5. Follow automated audible prompts.
6. Accessories are in the side bag (gloves, scissors, razor, etc).
7. Remove shirt (cut off if needed; shave hairy chest if needed).
8. Open CPR-D-padz pads packet and apply as shown on the packet (affix per defib diagram). If pad does not stick do not reuse. Use a new pad.
9. Device will assess victim for a heartbeat.
10. The device automatically determines if a shock is needed.
11. If indicated, do not touch the victim, stand back, push the center button to deliver the shock. Continue CPR and follow automatic prompts until EMS arrives.

Maintenance of our Zoll AED Plus device (Gold Level Service)

- RAP (two) AED devices have been registered and will be maintained by AED Service America telephone no. (860) 970-3250.
- The devices are warrantied for 7 years. Each has a registration number on the tag. Service includes an annual on-site inspection, and battery change, as needed.
- The device will automatically run a low-level self-test once a week and extensive self-test once a month.
- If operating properly a GREEN check will appear in the window next to the handle. If there is a malfunction, a blinking RED X appears as well as an audible beeping.
- We should report the malfunction (and serial number) as soon as possible to AED Service America (860) 970-3250. A technician will be dispatched typically within 4 hours (from West Chester, PA) but no longer than 48 hours to correct the problem. Note that the audible beeping cannot be turned off.
- The only requirement by RAP is to inspect for a GREEN check in the display window and report to Jeff Bevan (Site Manager) if there is a RED X and beeping sound.
- RAP administrator (Jeff Bevan) and Board members can access the AED Service America portal to examine maintenance history for our devices.